

Eco-Safe Surgical Instrument Rust & Stain Remover

High performance rust, stain spot, deposit and scale remover

Highly concentrated cost-effective formula

Rapid results within minutes after application

Environmentally friendly biodegradable formulation



Product Description:

Eco-Safe Surgical Instrument Rust & Stain remover is a high performance and rapid acting multi-purpose rust, stain, spot, deposit and scale remover for use in multiple applications including stainless steel surgical instruments, case carts, basins and trays. Its highly concentrated formula means excellent cost savings.

A biodegradable formulation means it's safe for disposal and the environment.

It includes an eco-friendly stainless steel passivator system to remove and then

prevent rust and corrosion, while removing and helping to prevent future scale redeposition. Its rapid acting formula works within minutes after application.





Directions & Usage:

Add 500ml to 5 litres (10% mixture) of hot water (55°-80°C). For soaking instruments, place solution in a pyrex or stainless steel container and then place the well cleaned instruments in a stainless steel wire bottom instrument tray. Leave to soak for 15 to 30 minutes, or for scrubbing carts and trays, wear gloves, apply the solution, and scrub thoroughly with a hard bristle brush. Remove instruments and rinse well in hot water. Do not use on silver plates or chrome instruments. Do not mix with other disinfectants or detergents. Close lid tightly after use.

Product Features:

- Removes rust, spots, stains, and limescale
- High performance formula acts in minutes
- Highly economical concentrated formula
- Restores a "like new" finish to instruments
- Extend lifespan of instruments
- Environmentally friendly
- Excellent stain removal properties
- Restores movement to instrument joints
- Stainless steel passivator system
- Removes and prevents rust and pitting
- Eliminates limescale
- Multiple use applications- ultrasonic baths, automated endoscope washers, disinfectors, instrument washers, immersion baths and conventional hand washing.

Contact Us

Safe Waste Solutions
087 057 7299

062 276 9104

info@safewaste.co.za

Visit us on the Web:

www.safewastesolutions.co.za

Shelf Life, Storage & Packaging:

Stable for 2 years. Store at ambient temperature out of direct sunlight. Undiluted pH = 1 -2. Appearance = Transparent – light yellow.

